

Amnicator®

A RAPID AND SIMPLE TO PERFORM SWAB TEST FOR THE DETECTION OF AMNIOTIC FLUID, INDICATING RUPTURED MEMBRANES

Contents: 100 peel pouches each containing 1 sterile individually wrapped Amnicator swab

STORE AT ROOM TEMPERATURE
AWAY FROM DIRECT SUNLIGHT

NO LATEX CONTAINED IN THIS PRODUCT / NO POLYVINAL CHLORIDE (PVL)

Medical Wire & Equipment Co (Bath) Ltd.

LEAFIELD INDUSTRIAL ESTATE CORSHAM WILTSHIRE ENGLAND SN13 9RT

Tel: +44 (0) 1225 810361 Fax: +44 (0) 1225 810153

Email: info@mwe.co.uk Internet: www.mwe.co.uk



Amnicator®

Test Procedure

1. Cleanse the vulva and vaginal introitus.
2. Carefully insert a bi-valve speculum (e.g. Cusco's speculum) into the vagina and sample exposing the cervix.
3. Pass an Amnicator into the vagina and sample for 5 seconds any liquor present at the external cervical os or in the posterior fornix of the vagina.
4. Withdraw the Amnicator and inspect the Colour of the swab tip after 5 seconds
5. Compare the colour of the Amnicator with the table.

Intact Membranes

pH 5.0 Yellow



pH 5.5 Yellow



pH 6.0 Yellow



Ruptured Membranes

pH 6.5 Blue-Green



pH 7.0 Blue



pH 7.5 Blue-Black



N.B. Antibiotic therapy or presence of a vaginal infection can alter the normal vaginal pH which may lead to false positive results



0088